## **HUGGA - Care & Cleaning Instructions**

Superficial dirt on the coating may be removed by wiping with a soft cloth moistened with water or neutral detergent and should be done so as quickly as possible after contact. More persistent contamination may be treated by wiping with alcohols or turpentine substitute, followed by hot water and detergent.

Cleaning and disinfection may be carried out on the coating with hand hot water and a neutral detergent or with a sodium hypochlorite solution (0.1%). **Abrasive cleaning agents should not be used.** 

Some surface wrinkling may result from cleaning procedures. This has no adverse effects on the fabric's properties.

All cleaning agents and disinfectants must be rinsed off and then dried before being used. Failure to do this may result in the accumulation of reagent that could damage the polyurethane coating and may negate the biocompatibility results of the Dartex fabric.

The fitted sheet can be washed up to 80 degrees centigrade. All other pieces are wipe clean only.

Store in a cool, dry place.